

## Contents

1. One Prosthesis
2. Two Prosthesis Covers
3. Two Pocketed Bras
4. One Prosthesis Holder
5. One Outer Packaging Kit

## Video

Watch our video on youtube illustrating instructions to use at

<https://www.youtube.com/watch?v=lj1ZSHMBQK8>


OR

[goo.gl/i6VfwS](http://goo.gl/i6VfwS)


## Contact Us

Please contact us for any further information


## Instructions to Use

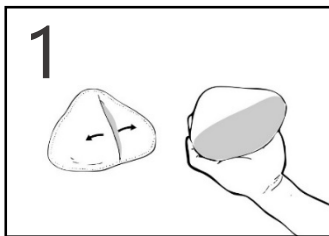


1 Create an opening in the fabric cover by pulling the two sides apart

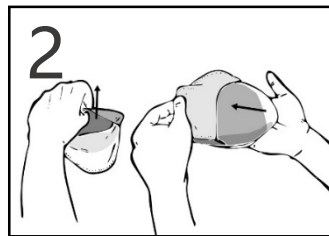


2 Gently insert top of prosthesis into the opening of the fabric cover towards the top of cover

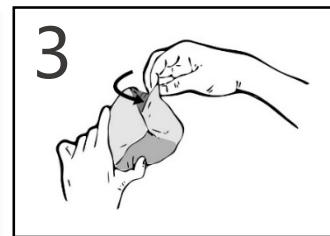


3 Pull the sides of cover to allow the prosthesis to slide into the cover completely

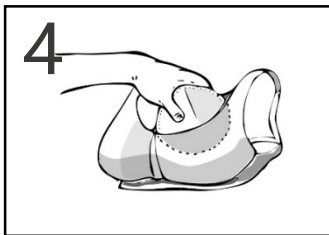

4 Place the fabric covered prosthesis into the pocket of bra

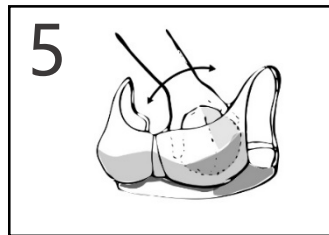

5 Gently rotate and move prosthesis to the desired location in the bra

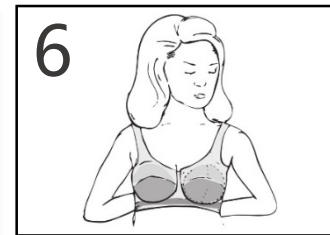

6 Now wear the pocketed bra containing the prosthesis.

*Note: Please read information on backside of this leaflet as well*

## Know Your Product

This product contains following items:

1. One Prosthesis – Manufactured in India using high quality medical grade materials imported from France, Germany and England. The prosthesis consists of silicone gel encapsulated by an outer polyurethane film. The materials are CE certified and tested to be biocompatible.
2. Two Prosthesis Covers – These are made of cotton and elastic fibers (Lycra) making it suitable to cover prosthesis for best fitting.
3. Two Pocketed Bras – High quality cotton and stretchable materials have been used in these bras to provide maximum comfort to users.
4. One Prosthesis Holder – Helps to maintain shape of prosthesis for longer period(s).
5. One Outer Packaging Kit – Made from waterproof synthetic material for compact packaging of the above six items.

## Maintenance

Don'ts:

1. Avoid direct contact with cosmetics (deodorants/perfumes/body cream) as they can cause damage to surrounding films.
2. Avoid sharp objects that may penetrate the skin of the breast prosthesis. Use special care with jewelry and long finger nails.
3. Avoid excessive squeezing, pulling, wringing or twisting of prosthesis as it can lead to permanent damage over time.
4. Avoid exposure of the prosthesis to very high temperature or direct sunlight.
5. Neither wash prosthesis in washing machine nor use scrubber to clean it.
6. Never use hot water to clean the prosthesis.
7. Don't create your own pocketed bra by stitching pockets as it can lead to bra slippage.
8. Underwire(s) or small sized bras can cause permanent damage over time to prosthesis.

Do's:

1. While not wearing, keep your prosthesis in the holder, provided in the box.
2. Always wear the prosthesis in the pocketed bra.
3. Be careful while placing prosthesis inside the bra so that it does not fall onto ground.
4. Gently wash prosthesis using lukewarm water and a very mild soap.
5. Allow the prosthesis to air dry or gently pat dry with soft towels.
6. Always hand wash the prosthesis cover and pocketed bras.

